



PUSH

FOR  
OUTSTANDING  
SMOOTH &  
CONSISTENT  
IOL DELIVERY

Vivinex™ multiSert™




HOYA  
SURGICAL OPTICS


## Vivinex™ multiSert™


## Designed for unmatched control at your fingertips

High user satisfaction amongst surgeons using Vivinex™ multiSert™<sup>1</sup>


An evaluation with 96 injectors by 14 European surgeons (*in vitro*) and 5 Japanese surgeons (*ex vivo* porcine eye) reported average usability and satisfaction levels between “Good” and “Excellent” for multiple facets of use. The refractive power of the evaluated IOLs was from +6.0 D to +29.0 D.

 *in vitro* (n=56)


 *ex vivo* (n=40)

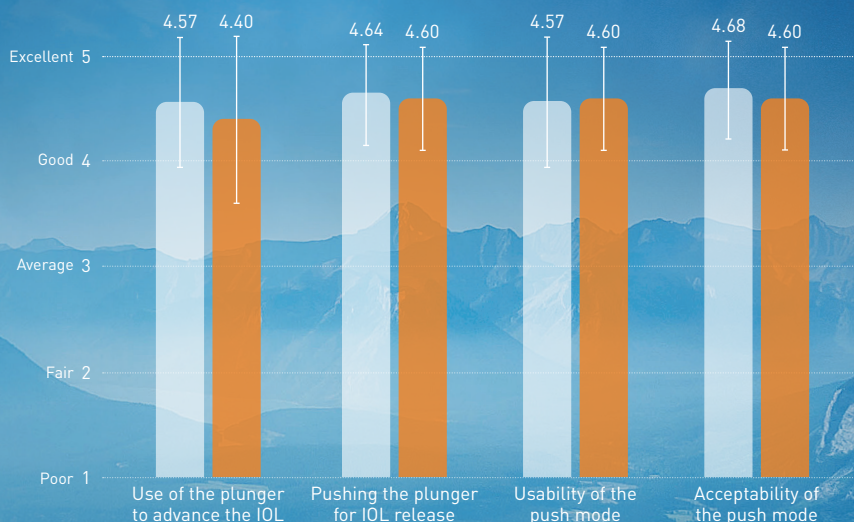multiSert™ has an uniformly low resistance force when delivering the Vivinex™ IOL in push mode, no matter what diopter<sup>2</sup>

## A comparison of resistance force at two different diopters

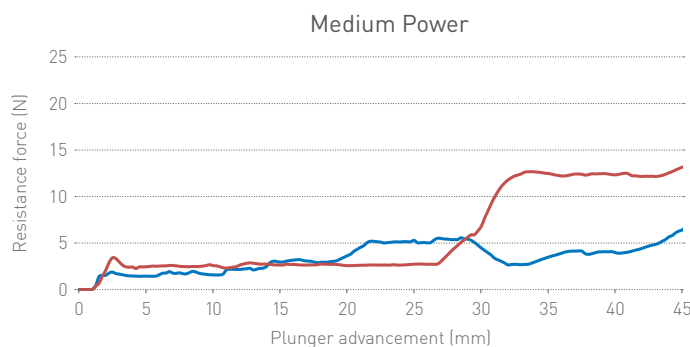

— +20.0 D Vivinex™ multiSert™ XY1-SP (n=10)  
— +20.0 D AcrySof IQ UltraSert AU00T0\* (n=3)

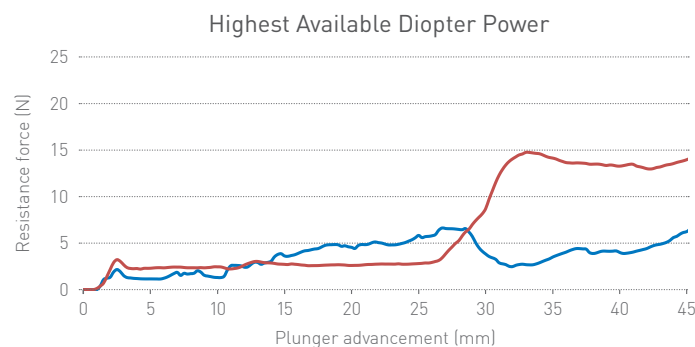

— +30.0 D Vivinex™ multiSert™ XY1-SP (n=10)  
— +27.0 D AcrySof IQ UltraSert AU00T0\* (n=3)

1. HOYA data on file. DoF-SERT-102-MULT-03052018, HOYA Medical Singapore Pte. Ltd. 2018. 2. HOYA data on file, HOYA Medical Singapore Pte. Ltd. 2018.

\*Third-party trademarks used herein are the property of their respective owners.

Information contained is intended for health care professionals. For a full list of indications and contra indications please refer to the Instructions For Use. Some of the products and/or specific features as well as the procedures featured in this document may not be approved in your country and thus may not be available there. Design and specifications are subject to change without prior notice as a result of ongoing technical development. Please contact our regional representative regarding individual availability in your country. HOYA, Vivinex and multiSert are trademarks of the HOYA Corporation or its affiliates. ©2020 HOYA Medical Singapore Pte. Ltd. All rights reserved.

 HOYA Medical Singapore Pte. Ltd | 455A Jalan Ahmad Ibrahim | Singapore 639939

 HOYA Surgical Optics GmbH | De-Saint-Exupéry-Straße 10 | 60549 Frankfurt am Main | Germany  
Hotline DE: Tel. +49 (0)800 664 2 664 | Fax +49 (0)800 774 2 774 [hoyasurgicaloptics.com](http://hoyasurgicaloptics.com)



0123 2020-12-15\_HSOE\_XC1-SP\_XY1-SP\_Insert\_EN

**HOYA**  
SURGICAL OPTICS