
## 🕅 Malyugín Ríng\_


## Ring Placement

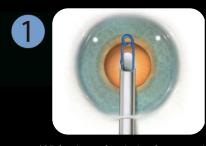


With viscoelastic in the anterior chamber inject the Ring starting with the cannula end at mid pupil



Just lift to disengage before placing the proximal scroll onto the iris

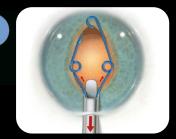


Inject while simultaneously retracting the injector. "Toeing" the injector down eases placement of the lateral scrolls

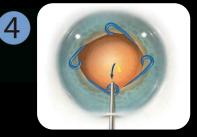


Use a Osher/Malyugin Ring Manipulator to place the proximal scroll




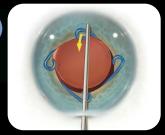

Use a Osher/Malyugin Ring Manipulator to disengage the distal scroll

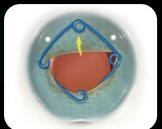


Place the distal scroll onto anterior iris

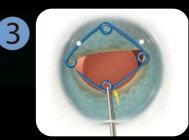


Move the proximal scroll toward the center of the pupil, and a bit anterior to disengage



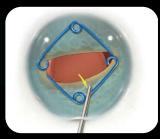


Place the proximal scroll onto anterior iris

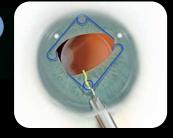

5

Simply enter the proximal scroll and pull by moving the button back

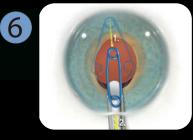

Stop when the left scroll rides above the cannula, then remove the "assembly"