VITREAL® S

**Triamcinolone Acetonide
Sospensione oftalmica 4,0%**

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EN

Indications

Medical device used in endocular surgery to stain the vitreous body during vitrectomy. The micro particles of Triamcinolone cause the vitreous humour to become opaque and make it easier to remove.

Instructions for use:

Shake the glass phial well before taking up the amount required. Use strictly aseptic methods during **VITREAL®S** as it contains no preservatives. Inject into the vitreous humour after correct prep of the operating area. Use maximum 30 G needle. Discard the phial and syringe with any remaining contents.

Composition

Micronised triamcinolone acetone 4.0%, monobasic sodium phosphate, dibasic sodium phosphate, sodium chloride, purified water. Does not contain preservatives

Precautions

- Product to be used exclusively for ophthalmic surgery
- Single-use pack
- Discard contents of the product must not be utilised
- Discard the phial and syringe with any remaining contents after use
- Product not for intravenous or intramuscular use; exclusively for intravitreal use
- Do not utilise the product after the expiry date indicated on the package
- Keep the product away from sources of heat, direct light and from children's reach
- Do not swallow
- Do not utilise in cases of known hypersensitivity towards any one of the product components
- Do not use if packaging is damaged
- Use and store the product at room temperature

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